


Basic Wheelchair Measurement Guide

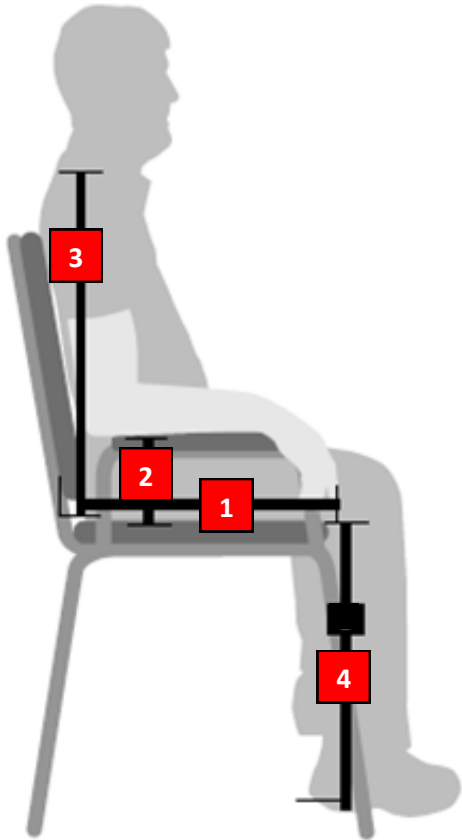

1) Seat Size

- Seat width should be determined by measuring from hip to hip in a straight line, then add two inches.
- Seat depth should be determined by measuring from the back of the hip to the back of the knee of the user while seated. Subtract one inch from this measurement.

Hip to Hip: _____ "

Hip to Knee: _____ "

2) Arm Height

Elbow to Seat: _____ "

3) Back Height

Collar Bone to Seat: _____ "

4) Floor to Seat Height

- Determine if user will need to use their feet to propel or move. Measure from back of knee to heel

Knee Back to Heel: _____ "

5) Wheelchair Weight and Weight Limit

Weight: of User: _____ LB